A black silhouette of a woman's face in profile on the left, looking towards the right. Her right hand is held palm-up, holding a dental bur. The background is white.

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# 0607

# CONTENT

|         |   |
|---------|---|
| 6 - 7   | HISTORY AND ISO   |
| 8 - 11  | NITI-TEE® FILES   |
| 11      | NiTi-TEE® Rotary files and NiTi-TEE® Rotary Taper 2 instruments |
| 12 - 17 | HAND FILES  |
| 14      | NiTi S-files and NiTi K-files                                   |
| 15      | NiTi K-reamers and NiTi H-files                                 |
| 16      | S-files and K-files in stainless steel                          |
| 17      | K-reamers and H-files in stainless steel                        |
| 18 - 30 | INSTRUMENTS AND ACCESSORIES                                     |
| 20      | Tri Auto ZX and GPR Gutta Percha Removing System                |
| 21      | WD-73 M / WD-74 M and WA-62 LT                                  |
| 22      | Barbed broaches   |
| 23      | K-finders and S-finders   |
| 24      | Finger pluggers and finger spreaders                            |
| 25      | Gates Glidden and Peeso reamers                                 |
| 26      | Extractor Del Zotto   |
| 27      | Endogrips and multigrips  |
| 28      | Standard Gutta Percha points and Gutta Percha for NiTi-TEE®     |
| 29      | Paper points and paste fillers                                  |
| 30      | File organizer  |
| 31      | STERILIZATION OF SENDOLINE INSTRUMENTS                          |

# ÍNDICE

|         |   |
|---------|---|
| 6 - 7   | <i>HISTORIA E ISO</i>   |
| 8 - 11  | <i>LIMAS NITI-TEE®</i>  |
| 11      | <i>Limas rotatorias NiTi-TEE® e Instrumentos Rotarios NiTi-TEE® Taper 2</i> |
| 12 - 17 | <i>LIMAS MANUALES</i>   |
| 14      | <i>Limas NiTi tipo S y limas NiTi tipo K</i>                                |
| 15      | <i>Ensanchadores NiTi tipo K y limas NiTi tipo H</i>                        |
| 16      | <i>Limas S y limas K de acero inoxidable</i>                                |
| 17      | <i>Ensanchadores K y limas H de acero inoxidable</i>                        |
| 18 - 30 | <i>INSTRUMENTOS Y ACCESORIOS</i>  |
| 20      | <i>Tri Auto ZX y Sistema GPR (Gutta Percha Removing System)</i>             |
| 21      | <i>WD-73 M / WD-74 M y WA-62 LT</i>   |
| 22      | <i>Tiranervios</i>  |
| 23      | <i>Limas localizadores: K y S</i>   |
| 24      | <i>Condensadores verticales y espaciadores</i>                              |
| 25      | <i>Taladros "Gates Glidden" y ensanchadores tipo Peeso</i>                  |
| 26      | <i>Extractor Del Zotto</i>  |
| 27      | <i>Endogrip y multigrip</i>   |
| 28      | <i>Puntas estándares de gutapercha y gutapercha para NiTi-TEE®</i>          |
| 29      | <i>Puntas de papel y léntulos</i>   |
| 30      | <i>Organizador de limas</i>   |
| 31      | <i>ESTERILIZACIÓN DE LOS INSTRUMENTOS SENDOLINE</i>                         |

# MANUAL OR MECHANICAL?

THE BEST OF BOTH WORLDS IN YOUR HANDS

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## HISTORY

Dental technician John Sjöding founded Sendoline in 1917. Having initially concentrated on manufacturing dental gold alloys and dental laboratory products, he soon realised the need to extend the scope of the operations to include endodontic products. His vision was to make endodontics simple, more ergonomic and patient friendly, which has since then remained at the core of Sendoline's focus.

One of Sendoline's most important product innovations is the H-file, which was developed in the 1940's in collaboration with the Swedish doctor Gustav Hedström. The launch of the H-file with its improved cutting ability introduced a new and more effective way for dentists to work. The H-file is still one of the most widely used files around the world today.

In 1980 the S-file was introduced. The cylindrical core design of the S-file provides good flexibility and excellent debris transportation. In addition, the double cutting edges provide an even greater cutting ability than the traditional H- or K-files.

The S-file is simplistically described as a double H-file, and is yet another example of how Sendoline has been able to contribute to the improvements within the endodontic field.

In 1990 Sendoline launched its first Nickel Titanium files along with the NiTi-TEE® Rotary file system, used for mechanical cleaning. The unique features of this material are flexibility, safety and durability – major innovations for both dentists and patients.

In 2005 Sendoline AB was acquired by W&H, one of the dental industry's oldest and most respected dental companies. With a growth excelling most rivals, Sendoline is rapidly making its footprint in the global dental market by being represented in over 50 countries.

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SENDOLINE – ISO

Our quality management system complies with ISO 9001:2000, ISO 13495:2003 and Annex II of MDD 93/42/EEC and all medical devices are CE-marked accordingly.

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# NITI-TEE<sup>®</sup> ROTARY FILES

A SMALL REVOLUTION THAT MAKES A BIG DIFFERENCE

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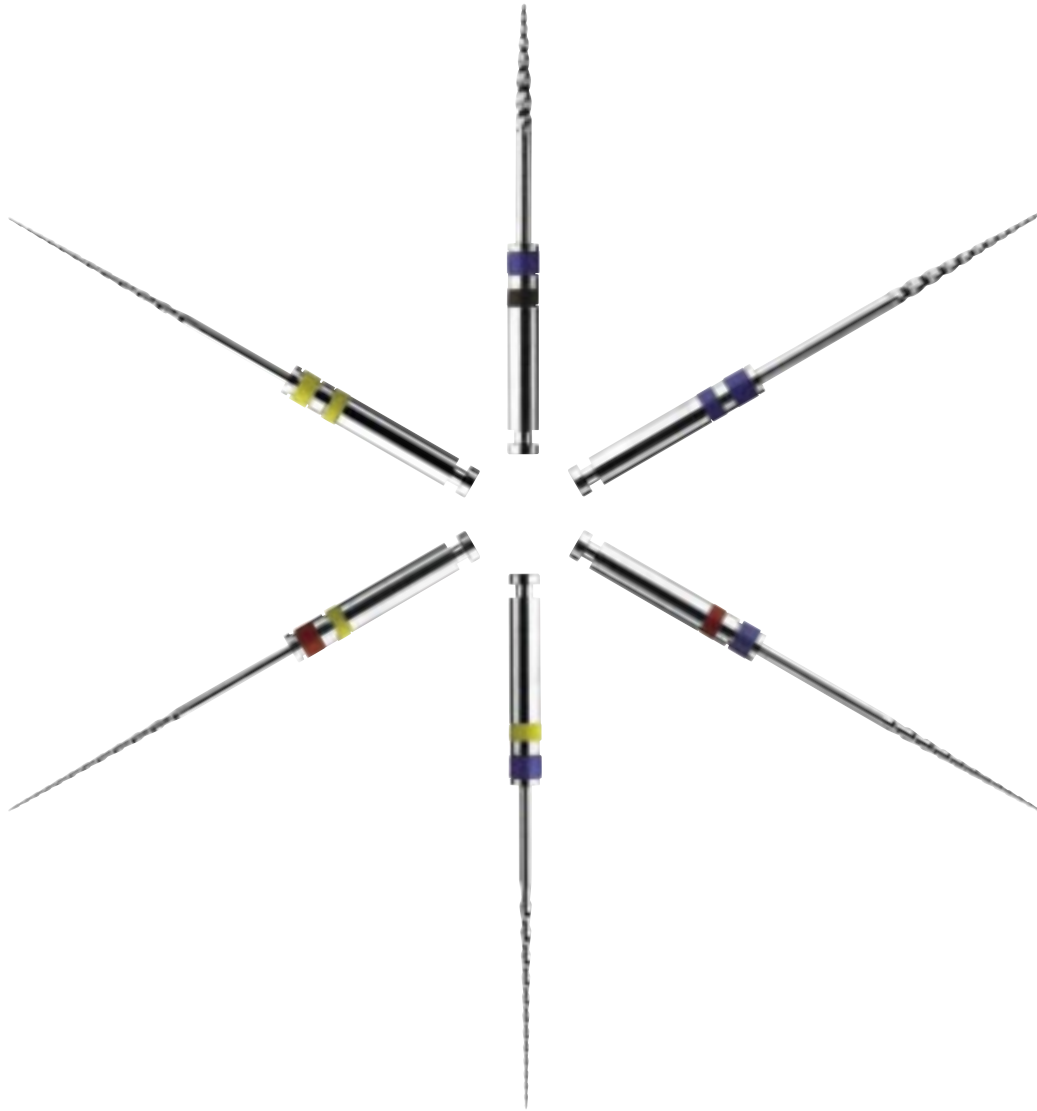
#### SENDOLINE NICKEL TITANIUM FILES

The Sendoline Nickel Titanium (NiTi) files consist of 54% nickel and 46% titanium.

All Sendoline Nickel Titanium files are designed with an enhanced safety tip, which allow the files to follow severely curved canals easily minimizing perforations, ledgings and zipping.

The unique characteristics of all NiTi-files are their improved:

- **Flexibility:** the cylindrical core gives the files high flexibility and excellent fracture resistance. In addition, the metal has memory, meaning that it returns back to its original position after being bent.
- **Safety:** the unique safety tip of the file prevents the "corkscrew effect" and makes it safe to use both for the dentist and the patient.
- **Durability:** the files are not affected by sterilization or by oral fluids and can be used 2-3 times more than stainless steel.

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#### NITI-TEE® ROTARY FILES

NiTi-TEE® tapered file system: the system has been developed for modern mechanical preparation using Crown Down technique.

Unlike the traditional hand file systems, the NiTi-TEE® tapered file system uses only 6 files, designed with different "tapers". The first three files, the 12/30, 8/30 and 6/30 are mainly used as "coronal shapers". The last three files in this new system, the 4/30, 4/25 and 4/20, have the unique "S"-profile, specially designed for mechanical apical preparation.

The files are designed to be used with endodontic rotating hand pieces with a speed of 300 rpm., including: W&H WD-74 M, Tri Auto ZX.

Sizes / ISO: 20, 25 and 30.

Length: 23 mm (File no. 12/30, 17 mm).

Package: introkit 6 pcs/box, individual refill 6 pcs/box.

Sterilization: see page 31.

#### NITI-TEE® ROTARY TAPER 2 INSTRUMENTS

NiTi-TEE® Rotary Taper 2 instruments are used for widening the apical part of the canal up to ISO size 55, using Standardized technique. The root canal is shaped with tapered instruments which give the root canal its optimal form in order to facilitate further treatment. However, some root canals may require a wider apical cleaning than that which a normal Crown Down set of files can offer. NiTi-TEE® Taper 2 refill enables apical cleaning up to ISO size 55.

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UALES NITI  
NTA DE LOS DEDOS

#### THE NITI S-FILES - THE UNIVERSAL FILE

The cylindrical core design provides good flexibility and excellent debris transportation, while at the same time the double cutting edges provide great cutting ability. The unique spiral design flutes successively increase in depth and length from tip to handle and double flutes at the 90° non-cutting tip makes the instrument efficient but yet safe to use.

The NiTi S-file has an ergonomic handle which is less tiring and increases the tactile feel.

Sizes / ISO: 15-80.  
Length: 21, 25, 28 and 31.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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#### NITI K-FILES

The NiTi K-file is designed in a tight spiral that has four cutting edges with a 60° tip. Its great flexibility and good cutting ability makes it safe to use.

Sizes / ISO: 15-80.  
Length: 21, 25, 28 and 31.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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### NITI K-REAMERS

Unlike the NiTi K-file, the K-reamer has a loose spiral design with three cutting edges. The great advantage of this instrument is the excellent debris transportation. The instrument has a 60° tip.

Sizes / ISO: 15-80 .  
Length: 21, 25, 28 and 31.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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### NITI H-FILES

The NiTi H-file has one cutting edge with a 90° angle, with its unique spiral design flutes which it successively increase in depth and length from tip to handle. In addition, the cylindrical core design makes the Sendoline NiTi H-file one of the most flexible and efficient, thus one of the most popular H-files on the market today.

Sizes / ISO: 15-80.  
Length: 21, 25, 28 and 31.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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# SWEDISH STAINLESS SANDVIK STEEL

Stainless Swedish steel from Sandvik is characterised by its good flexibility and excellent corrosion resistance, assuring the consistent high quality of hand files within the product portfolio of Sendoline.

## S-FILES

The characteristic double edges give the S-file its excellent cutting ability. The construction of the cylindrical core has been designed to be more flexible while enduring increased tension, hence minimizing the overall risk of fractures. In addition, it comes with a non-cutting tip which increases safety.

Its ergonomic handle and excellent cutting ability further minimizes the total working time.

Sizes / ISO: 06-140.  
Length: 21 mm, 25 mm, 28 mm and 31 mm.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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## K-FILES

The flexibility of the K-file is excellent and is safe to use with its 60° tip. Its tight spiral design with four cutting edges provides good cutting ability.

Sizes / ISO: 06-140.  
Length: 21 mm, 25 mm, 28 mm and 31 mm.  
Package: individual or assorted sizes 15-40 or 45-80.  
All packages come with 6 pcs/box.  
Sterilization: see page 31.

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## K-REAMERS

Compared to the K-file the reamer has an open spiral design with three cutting edges with a 60° tip. The great advantage of this instrument is the excellent debris transportation.

Sizes / ISO: 06-140.

Length: 21 mm, 25 mm, 28 mm and 31 mm.

Package: individual or assorted sizes 15-40 or 45-80.

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## H-FILES

The cutting ability of the H-file is very good and the cylindrical core design makes the Sendoline H-file one of the most flexible H-files on the market today. It is designed with "one" cutting edge with a 90° angle. Its unique spiral design flutes successively increase in depth and length from tip to handle, enhancing its efficiency while maintaining its safety.

Sizes / ISO: 06-140.

Length: 21 mm, 25 mm, 28 mm and 31 mm.

Package: individual or assorted sizes 15-40 or 45-80.

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PRODUCTS  
POSSIBILITIES

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#### TRI AUTO ZX

The Tri Auto ZX is used for mechanical cleaning of root canals, and integrates an electronic high precision torque control and apex-locator in one ergonomic and cordless hand piece.

The major advantages of the Tri Auto ZX:

- Minimizes risk for file fractures with automatic torque control
- Provides a complete solution with the in-built Apex locator
- Provides total portability with cordless endo motor
- Compatible with any mechanical file that complies with the ISO requirements
- Easily re-chargeable with the docking station provided
- Fully charged battery lasts for one hour's continuous use

Sterilization: see page 31. (The motor, "the tip" of the Tri-Auto is removable and autoclavable. The body, "the white part" of the Tri Auto should be carefully cleaned with alcohol).

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#### GPR-GUTTA PERCHA REMOVING SYSTEM

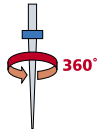
The new GPR system is developed to remove old filling material from the root canal. The GPR instruments are used with Crown Down technique.

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*Tamaños / ISO: 1, 2, 3, 4, 5.  
Envase: 5 un./caja.*

#### WD-73 M AND WD-74 M

The WD-73 M / WD-74 M is used for mechanical cleaning of root canals. The small heads improve visibility and permit an excellent view along with enhanced accessibility. Their ergonomic design and ease of maintenance assure optimum hygiene.

The WD-73 M has a reduction ratio of 70:1 and is used for motors with a speed of 25,000 rpm, (e.g. air motors).

The WD-74 M 128:1 used for motors speed 40,000 rpm (e.g. electric motors).

Sterilization: see page 31. We recommend sterilization conforming to EN 13060 class B cycle at 134°C and 3 minutes minimum holding time. Please respect national recommendations if applicable.

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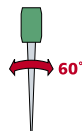
#### WA-62 LT

The WA-62 LT (ENDO Cursor) is used for mechanical cleaning of root canals. This instrument has a reduction ratio of 4:1. The root canal is prepared by means of a reciprocating movement of the instrument (approx 60 degrees); this, in combination with the integrated spray, provides efficient cleaning of the files during use.

The WA-62 LT also has powerful illumination which delivers light directly to the operative site. The unique, extremely robust compact glass rod delivers a very precise light, even after several thousand sterilization cycles.

Sterilization: see page 31. We recommend sterilization conforming to EN 13060 class B cycle at 134°C and 3 minutes minimum holding time. Please respect national recommendations if applicable.

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#### BARBED BROACHES

The barbed broaches are designed to remove pulp tissue efficiently. The instrument is introduced into the canal, rotated 180 degrees to pull the pulp tissue out.

Sizes / ISO: 0, 1, 2, 3, 4, 5, 6.

Length: 21 mm.

Package: individual or assorted sizes 1-6.

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#### K-FINDER

The K-finder is specifically developed for mechanical cleaning. It is used to locate difficult canals as well as creating a "glide path" to enable the mechanical instruments to work efficiently and effectively.

Sizes / ISO: 15.

Length: 21 mm.

Package: individual sizes. All packages come with 6 pcs/box.

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#### S-FINDER

The special features of the S-finder, the double cutting edges and rigidity, make it optimal for locating the canal.

Sizes / ISO: 10.

Length: 18 mm and 21 mm.

Package: individual sizes. All packages come with 6 pcs/box.

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#### FINGER PLUGGERS

The pluggers facilitates the insertion of the gutta percha points. It has a flat tip and a 2% taper. The instrument is used for vertical condensation of gutta percha points.

Sendoline offers pluggers in stainless steel and in NiTi.

Sizes / ISO: 15-80.

Length: 25 mm.

Package: individual sizes or assorted sizes 15-40 or 20-80.

Assorted come with 6 pcs/box and refills with 2 pcs/box.

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#### FINGER SPREADERS

The spreader has a conical tip and is designed for lateral condensation of gutta percha points.

Sizes / ISO: 15-80.

Length: 25 mm.

Package: individual sizes or assorted sizes 15-40.

Assorted come with 6 pcs/box and refills with 2 pcs/box.

Material: available in stainless steel and in NiTi.

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#### GATES GLIDDEN

The gates drills are used to enlarge the upper part of the canal.

Sizes / ISO: 1-6.

Length: 19 mm.

Package: individual sizes or assorted sizes 1-6.

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#### PEESO REAMERS

The Peeso reamers enlarge the canal and prepare the canal entrance. The Peeso can also be used for removal of Gutta Percha to create post space.

Sizes / ISO: 1-6.

Length: 19 mm.

Package: individual sizes or assorted sizes 1-6.

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#### EXTRACTOR DEL ZOTTO

The unique and flexible spiral with its user friendly design facilitates an easy and safe removal of fractured instruments in straight or bent root canals.

1. Loosen the fragment, then insert the extractor into the root canal and twist it 180 degrees clockwise; the spring opens and can be pulled over the instrument.
2. Twist the extractor anti-clockwise with a pulling motion, slowly removing the fragment from the canal.
3. The instructions on the CD should be carefully studied before use.

Sizes / ISO: 20-60 (The ISO size indicates the smallest inner diameter of the instrument).

Length: 25 mm.

Package: individual sizes or assorted sizes 20-60. 6 pcs/box.

Sterilization: see page 31. The Extractor is for single use only but can be sterilized).

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#### ENDOGRIP

Endogrip is a sleeve made of sterilizable silicone rubber giving Sendoline files a thicker handle. This is particularly useful when working with gloves. Endogrip comes in 4 colours for identifying the different file types. The Endogrip design enhances the tactile feel.

Sizes / ISO: fits most files on the market, but are optimized for Sendoline files.  
 Package: assorted 100 pcs box and individual packs of 10 pcs/box.  
 Sterilization: see page 31.

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#### MULTIGRIP

Multigrip is an instrument used as an aid in conjunction with endodontic treatments. It is made of aluminium and can be used with files or finders to locate the root canal, or with other instruments, such as interdental brushes and wedges, when reaching parts of the oral cavity.

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#### STANDARD GUTTA PERCHA POINTS

Gutta Percha points used for root canal filling after cleaning the canal with taper 2 instruments.

Sizes / ISO: 15-100.

Package: individual sizes or assorted sizes 15-40 or 45-80. In addition to the individual sizes, X-Fine, Fine-Fine, Medium-Fine, Fine, Fine-Medium, Medium, Medium-Large and Large can also be ordered. 120 pcs/box.

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#### GUTTA PERCHA FOR NITI-TEE®

The Sendoline tapered Gutta Percha for NiTi-TEE® simplifies the root canal filling. It is precise, rapid and easy to use with only one Gutta Percha point required.

Used for root canal filling after cleaning the canal with NiTi-TEE® Rotary Files.

Sizes / ISO: 15-40.

Tapers: 4 and 6.

Package: individual sizes or assorted sizes 15-40. 60 pcs/box.

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#### PAPER POINTS

Paper points for absorption of liquids during root canal treatments. The Sendoline Paper Points are especially rolled to exact dimensions optimizing the root canal treatment with its excellent absorbance ability.

Sizes / ISO: 15-100.

Package: individual sizes or assorted sizes 15-40 and 45-80. 150 pcs/box.

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#### PASTE FILLERS

The Sendoline paste filler is a motor driven taper 2 instrument available in four different sizes. The length and thickness of the canal determines which instrument is to be selected. The instrument is filled with paste and inserted into the canal. It is then slowly pulled out of the canal with a rotating motion.

Sizes / ISO: 25-40.

Length: 21, 25, 29 mm.

Package: individual sizes or assorted sizes 25-40.

All packages come with 4 pcs/box, with or without spring.

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#### FILE ORGANIZER – (STAINLESS STEEL BOX)

Sterilizable file organizer in stainless steel. The box has a file usage indicator and a millimetre scale for conventional measuring of the root canal.

The organizer has room for up to 8 instruments.

Use: file organizer.

Sizes / ISO: compatible with most files on the market.

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## STERILIZATION OF SENDOLINE INSTRUMENTS

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All Sendoline instruments are non-hollow, non-porous loads and can be sterilized in a normal gravity-process sterilizer at 134 degrees Celsius, at a minimum holding time of 3 minutes.

However, in order to render the instruments sterile and keep them sterile to the point of use, all items must be wrapped. In order to safely sterilize and dry wrapped loads, it is strongly recommended that sterilization conform to a class B cycle according to EN 13060.

Sendoline recommends the LISA® sterilizer from W&H Sterilization.

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